## Indications for use

Uni or bilateral breast surgery, reduction, augmentation (prostheses and/or fat), reconstruction (with the exception of free flaps such as Diep, Gracilis, Pap) and after partial ablation (tumorectomy) or total (mastectomy), pedicled flaps. Mastodynia on fibrocystic mastosis and/or pre-menstrual, breast engorgement, breast infections (in addition to treatment antibiotic and local usual). Gynecomastia surgery associated or not with peripheral liposuction.

## Contraindications

Breast reconstruction by free flap (Diep, Gracilis, Pap), microsurgical reconstructions, cryoglobulinemia, allergy to cold, insensitivity to cold, arteritis, diabetes.

## Risks of use and precautions for use

This medical device must be used according to the medical prescription of your doctor or according to the recommendations for use of your duly authorized health professional (pharmacist, physiotherapist, orthotist, etc.).

The manufacturer makes no warranty that use of this product will not prevent injury. It is imperative to check the completeness and integrity of the elements of the kit before use. Do not use if damaged.

Factors can increase the risk of skin burns (malnourished patients, excessive smoking, insulin-dependent diabetics, or even people with fragile skin) : close medical supervision is recommended.

Cold packs should never be in direct contact with the scar nor skin. Place the cold packs over the compression bra, or use a protective fabric. Cold packs should be soft and frost free before use.

Do not heat or warm cold packs.

In case of embarrassment of discomfort, remove the cold packs.

Do not pierce the cold packs.

In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drainage system connected to the sewers, or into a watercourse.

Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center.

In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water.

Contact your doctor if irritation persists.

Keep out of the reach of children and people unfit to use this device.

## Cleaning

Cold packs are hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath.

If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces).

Leave to air dry and lay flat.

For more information, contact Orthonov or visit the website [www.orthonov.com](http://www.orthonov.com)

### Warnings

This medical device is a regulated health product which follows under this regulation, the CE marking.

Before use, please read this manual carefully and follow the recommendations for use.

In the event of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your authorized healthcare professional as soon as possible (pharmacist, physiotherapist, orthotist, etc.), and contact

Orthonov: 01 60 13 65 73 or [www.orthonov.com](http://www.orthonov.com)

In the event of a serious reaction, contact the distributor and the competent authority in France.

Medical Device

CE Mark

Product traceability

Single patient multiple use

Batch Code

Use-By Date

Temperature limit

Consulte instructions for use

WARNING  
Manufacturer Caution,  
Consult Accompanying Documents

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