Indications for use

Medical gynecology: indication of pain in women except pregnant women
Gynecological surgery: indication of pain in women except pregnant women
Visceral and bariatric surgery (obesity): laparoscopy (or laparascopy), eventration, celioscopy.
Spine surgery: ALIF (anterior arthrodesis), OLIF (lateral approach), TFIL (minimally invasive posterior lumbar arthrodesis), PLIF (posterior lumbar interbody fusion), spine trauma.
Thoracic surgery: chest fracture, rib fracture or rib cracks enveloping the thorax, trauma to the spine, pleuroscopies, thorascopies, mediastinoscopies.
Cosmetic surgery: abdominoplasty, dermolipectomy.

Contraindications

Allergy to cold, insensitivity to cold, Raynaud's syndrome or other vasospastic diseases, arteritis, cryoglobulinemia, sickle cell disease, arteriosclerosis, diabetes, polyarthritis, disorders of skin sensitivity or hypercoagulation, peripheral vascular diseases causing ischemia or weak local blood circulation.

Risks of use and precautions for use

This medical device must be used according to the medical prescription of your doctor or according to the recommendations for use of your duly authorized health professional (pharmacist, physiotherapist, orthotist, etc.). The manufacturer makes no warranty that use of this product will not prevent injury. It is imperative to check the completeness and integrity of the elements of the kit before use. Do not use if damaged. Factors can increase the risk of skin burns (malnourished patients, excessive smoking, insulin-dependent diabetics, or even people with fragile skin) : close medical supervision is recommended. Cold pack should never be in direct contact with the scar nor skin. Use a protective fabric. Cold pack should be soft and frost free before use. Do not heat or warm cold pack. In case of embarrassment of discomfort, remove the cold pack. Do not pierce the cold pack. In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drainage system connected to the sewers, or into a watercourse. Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center. In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water. Contact your doctor if irritation persists. Keep out of the reach of children and people unfit to use this device.

Cleaning

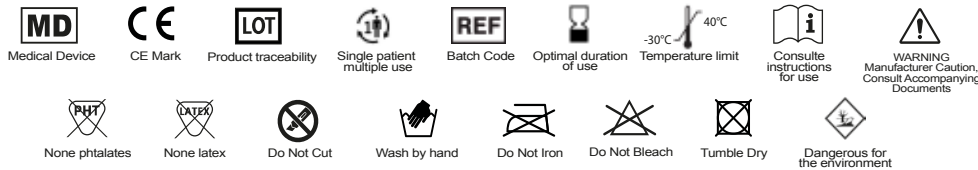
Cold pack is hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath. If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces). Leave to air dry and lay flat. For more information, contact Body Icing or visit the website www.bodyicing.com

Warnings

This medical device is a regulated health product which follows under this regulation, the CE marking. Before use, please read this manual carefully and follow the recommendations for use. In the event of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your authorized healthcare professional as soon as possible (pharmacist, physiotherapist, orthotist, etc.), and contact Body Icing: 0801 790 000 or www.bodyicing.com. In the event of a serious reaction, contact the distributor and the competent authority in France.

Body Icing Manufacturer
 27 Rue Jacques Lemercier - 78000 Versailles
 Email : contact@bodyicing.com
N°Vert 0801 790 000
 APPEL GRATUIT

Body Icing Distributor
 27 Rue Jacques Lemercier - 78000 Versailles - France
 Email : contact@bodyicing.com
N°Vert 0801 790 000
 APPEL GRATUIT

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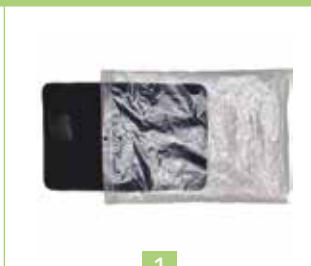
Place the cold pack in the airtight bag, then put all in the freezer (between -18 and -22°C) for 8 hours before use.

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If the cold pack is hard or has frost, wait for it to warm up. Never use directly on skin.

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Place the belt with the 2 central whales in the back (the 4 whales can be removed).

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Place the cold pack lengthwise, at the edge of belt (or in the back for spinal use).

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Cover the abdomen with the cold pack then close the belt with the velcros. Extension can be used if necessary.

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After 20 minutes, open the belt abdominal area, and remove the cold pack, then put it back in the bag and put it back in the freezer.

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It is advisable to carry out cryotherapy sessions of 20 to 30 minutes every 2 hours, avoiding direct contact with the skin. For any questions related to the use of the products, consult your doctor, surgeon or authorized health professional. (pharmacist, physiotherapist, orthotist, etc.), and contact BODY ICING on: 0801 790 000 or by email: contact@bodyicing.com

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