

**Cryotherapy kit for breast surgery
Breast Icing**



Contents

Réf. EST60-SEIN




INSTRUCTIONS



Place the back strap flat, on a square surface, with the Velcro pointing upwards (red arrows).



An extension cord is available for the back strap if required.

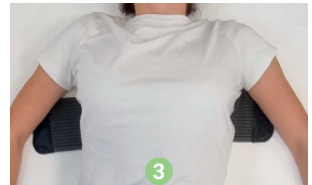


Lie on top of it, so that the back strap is at the same level as the breasts.



Place the cold pack of the right breast (the one without velcros), placing the areola in the center of the hole.



Place the other cold pack of the left breast, then fix the 2 velcro of the pack covering on the right breast.

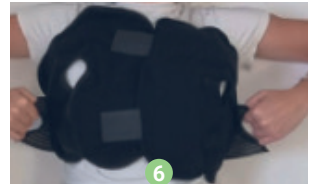

On each side, pass your fingers through the openings of the back strap, and fix it on each pack of cold.

 The Breast Icing brace should be used in a lying position, at the risk that the cold packs do not remain in place.

Indications for use

Uni or bilateral breast surgery, reduction, augmentation (prostheses and/or fat), reconstruction (with the exception of free flaps such as Diep, Gracilis, Pap) and after partial ablation (tumorectomy) or total (mastectomy), pedicled flaps. Mastodynia on fibrocystic mastosis and/or pre-menstrual, breast engorgement, breast infections (in addition to treatment antibiotic and local usual). Gynecomastia surgery associated or not with periphe-ral liposuction.

Contraindications

Breast reconstruction by free flap (Diep, Gracilis, Pap), microsurgical reconstructions, cryoglobulinemia, aller-gy to cold, insensitivity to cold, arteritis, diabetes.

Risks of use and precautions for use

This medical device must be used according to the medical prescription of your doctor or according to the re-commendations for use of your duly authorized health professional (pharmacist, physiotherapist, orthotist, etc.). The manufacturer makes no warranty that use of this product will not prevent injury. It is imperative to check the completeness and integrity of the elements of the kit before use. Do not use if damaged.

Factors can increase the risk of skin burns (malnourished patients, excessive smoking, insulin-de- pendent dia-betics, or even people with fragile skin) : close medical supervision is recommended.

Cold packs should never be in direct contact with the scar nor skin. Place the cold packs over the compression bra, or use a protective fabric. Cold packs should be soft and frost free before use.

Do not heat or warm cold packs.

In case of embarrassment of discomfort, remove the cold packs. Do not pierce the cold packs.

In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drai-nage system connected to the sewers, or into a watercourse.

Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center.

In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water. Contact your doctor if irritation persists.

Keep out of the reach of children and people unfit to use this device.

Cleaning

Cold packs are hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath.

If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces). Leave to air dry and lay flat.

For more information, contact Body Icing or visit the website www.bodyicing.com

⚠ WARNINGS ⚠

This medical device is a regulated health product which follows under this regulation, the CE marking. Before use, please read this manual carefully and follow the recommendations for use. In the event of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your authorized healthcare professional as soon as possible (pharmacist, physiotherapist, orthotist, etc.), and contact Body Icing:

+33 801 790 000 or www.bodyicing.com

In the event of a serious reaction, contact the distributor and the competent authority in France.



Medical device



CE Mark



Product traceability



Single patient multiple use



Batch Code



Use-By Date



-30°C 40°C
Temperature limit



Consulte instructions for use



WARNING
Manufacturer Caution,
Consult Accompanying
Documents



None phthalates



None latex



Do Not Cut



Wash by hand



Do Not Iron



Do Not Bleach



Tumble Dry



Dangerous for the environment